NOVELTIES IDS 2019

NEUHEITEN IDS 2019

MORSA CAD Wax gingiva - Prothetik

Article.Number: DR000732HD830M

CE

For making functional try ins and individual bite rims in the complete denture technique

The process supporting CAD discs are used to transform conventional working steps in the laboratory into complete digital processes.

The discs can also be used for conventional finishing procedures in injection and pressing techniques - the ability to boil it out for wax removal makes it possible

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MORSA Gingiva Extra Tough Sheet wax – new innovative formulation 500 g

Article.Number: DM000154VA804

A tough, four-seasons setting-up wax with high softening temperature. Used for setting up denture teeth in wax for all prosthetic restorations, e.g. equally suitable for partial and full acrylic dentures, chrome cobalt saddles, immediate and temporary dentures and extensions. The natural-looking gingival shading corresponds to hot and cold curing denture acrylics.

- High flow temperature enables longer try-ins and use of duplicating gels
- Minimum spring-back (flow)
- High shape and dimensional accuracy
- Low setting contraction, minimum warpage of the set-up after cooling

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Quick-Bite Innovative Orthodontic wax bite - **patented**

Article.Number : DBSH0001 20 Pieces per Package

- Innovative wax bite for use in splint therapy, sleep medicine and orthodontics. Patented by Prof. Godt / Schweizer, Dental Technician. Easy, reliable bite registration with defined occlusal height using integrated stops.
- Wax bite with defined occlusal height (2 mm) for use in orthodontics, sleep medicine, bruxism appliances and bite-raising appliances. Easy, reliable use. Using this wax bite your team will be able to take reliable, reproducible bite registrations, highly improved speed (quickly heated, quickly cooled). For reliable bite registration with orthodontic appliances, bruxism and bite-raising appliances, in sleep medicine.

How to use:

- The wax bite should be immersed in a 45°C warm water tank for 20-25 seconds before use. After insertion in the mouth and slight adaptation to the upper or lower dental arch, the lower jaw should be moved to the target position. The patient should then bite slowly together until she/he feels noticeable resistance and stops. The bite can then be removed and checked visually. The correct biting depth is identified by the colour change on the impressions.
- The wax bite can be used in the dental lab in the usual way.

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VDDI
Dental Solutions.
German Manufacturers.

MORE EFFICIENCY
WITH MORSA

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Morsa Wachswarenfabrik Sallinger GmbH