NOVELTIES IDS 2019

NEUHEITEN IDS 2019

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Innovative Orthodontic wax bite - patented

Article.Number : DBSH0001 20 Pieces per Package

- Innovative wax bite for use in splint therapy, sleep medicine and orthodontics. Patented by Prof. Godt / Schweizer, Dental Technician. Easy, reliable bite registration with defined occlusal height using integrated stops.
- Wax bite with defined occlusal height (2 mm) for use in orthodontics, sleep medicine, bruxism appliances and bite-raising appliances. Easy, reliable use. Using this wax bite your team will be able to take reliable, reproducible bite registrations, highly improved speed (quickly heated, quickly cooled). For reliable bite registration with orthodontic appliances, bruxism and bite-raising appliances, in sleep medicine.

How to use:

- The wax bite should be immersed in a 45°C warm water tank for 20-25 seconds before use. After insertion in the mouth and slight adaptation to the upper or lower dental arch, the lower jaw should be moved to the target position. The patient should then bite slowly together until she/he feels noticeable resistance and stops. The bite can then be removed and checked visually. The correct biting depth is identified by the colour change on the impressions.
- The wax bite can be used in the dental lab in the usual way.

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Morsa Wachswarenfabrik Sallinger GmbH